

Default eUnity Keyboard shortcuts

Action name	Shortcut
2D / 3D View Toggle (Mammography)	Shift + J
Angle Measurement	A
Apply Notification Action	H
Arrow	Y
CAD Marker (Mammography)	U
Circle ROI	C
Cobb Angle Measurement	B
Collapse / Expand Vertical Series Tray	Shift + F
Create / Delete Key Image	Ctrl + Space
Delete Selected Markup	Delete
Demographics	D
DICOM Header	Shift + D
Ellipse ROI	E
First Series Page	Shift + Page Up
Fit to Window (Mammography)	Shift + A
Free Rotate	Forward Slash
Freeform ROI / Doppler Envelope	F
Freehand	G
Fusion Blending (Advanced Visualization)	Shift + U
Help	F1
Horizontal Flip	Shift + H
Image Sharpening Preset 1	Shift + 1
Image Sharpening Preset 2	Shift + 2
Image Sharpening Preset 3	Shift + 3
Image Sharpening Preset 4	Shift + 4
Image Sharpening Preset 5	Shift + 5
Image Sharpening Preset 6	Shift + 6
Image Sharpening Preset 7	Shift + 7
Image Sharpening Preset 8	Shift + 8
Image Sharpening Preset 9	Shift + 9
Invert Brightness	I
Last Series Page	Shift + Page Down
Linear Measurement	L
Link Studies : Navigation / Zoom and Pan	K
Magnifying Glass Toggle	Alt + I
Markup Toggle	M
Mute Cine (Video)	Shift + Space

Action name	Shortcut
Navigate Mode	N
Navigate to First Image	Home
Navigate to Last Image	End
Next Image	Arrow Down
Next Image Sharpening Preset	Shift + S
Next Presentation Step (Hanging Protocol)	Shift + Arrow Right
Next Quadrant (Mammography)	Shift + Arrow Down
Next Relevant Prior	Alt + M
Next Series	Arrow Right
Next Series Page	Page Down
Next Study in Study List	Alt + K
Notification Panel	Shift + N
Oblique (Advanced Visualization)	Shift + O
Perpendicular Measurement	Shift + X
Play / Pause (Cine / Video)	Space
Point Tool	P
Previous Image	Arrow Up
Previous Presentation Step (Hanging Protocol)	Shift + Arrow Left
Previous Quadrant (Mammography)	Shift + Arrow Up
Previous Relevant Prior	Alt + N
Previous Series	Arrow Left
Previous Series Page	Page Up
Previous Study in Study List	Alt + L
Print	Ctrl + P
Quadrant View (Mammography)	Shift + Q
Ratio Measurement	J
Reference Lines	S
Remove Image Sharpening	Shift + O
Report / Comments	R
Reset Series (Multiphase)	Shift + Z
Revert to Original	O
Rotate Left	Shift + W
Rotate Right	Shift + C
Save Presentation State	Ctrl + S
Series Reposition	Q
Sort by Phases (Multiphase)	Shift + K
Spine Labeling	V
Split out Phases (Multiphase)	Shift + Y
Study List	Shift + L

Action name	Shortcut
Text Annotation	X
Triangulation	T
Vertical Flip	Shift + V
Window Level Mode	W
Window Level Preset 1	1
Window Level Preset 2	2
Window Level Preset 3	3
Window Level Preset 4	4
Window Level Preset 5	5
Window Level Preset 6	6
Window Level Preset 7	7
Window Level Preset 8	8
Window Level Preset 9	9
Window Level Reset	0
Zoom In	Equal Sign
Zoom Out	Dash